


LiquiMedLock™

Urine Transfer Device

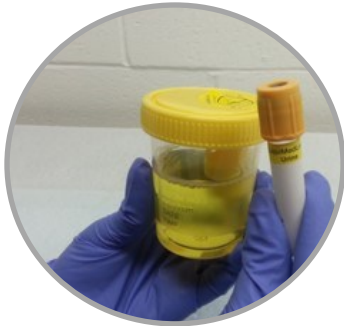

Ensure accuracy and minimize risk of sample contamination with the LiquiMedLock™ urine transfer device.

- Collection Cup with integrated Transfer Tube is quick and easy to use.
- Direct transfer of sample reduces errors and maintains efficient workflow.

Easy and quick to use

1



Collection Cup

Transfer Tube

Collect urine sample in the Collection Cup and label the Transfer Tube accordingly.

2




Peel off the top label seal of the Collection Cup.

3



Invert the Transfer Tube and place in the opening of the collection cup.

4

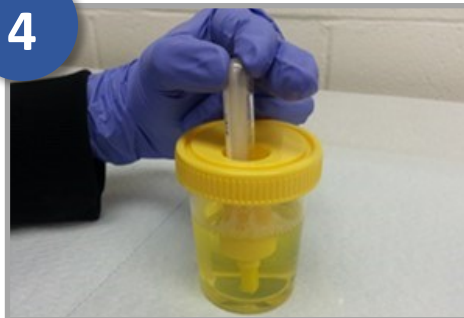

Press down on the Transfer Tube to create a vacuum suction. Then continue pressing down until sample fills the Transfer Tube.

5



Sample will fill the Transfer Tube in 2-3 seconds.

6



Analyse urine sample in the Transfer Tube.



- 1043 Pedigo Way, STE 15 Bowling Green KY 42103
- Phone: 270-904-5097 • Fax: 270-904-5864
- acculabtox.com